


EYE DROP SYSTEM

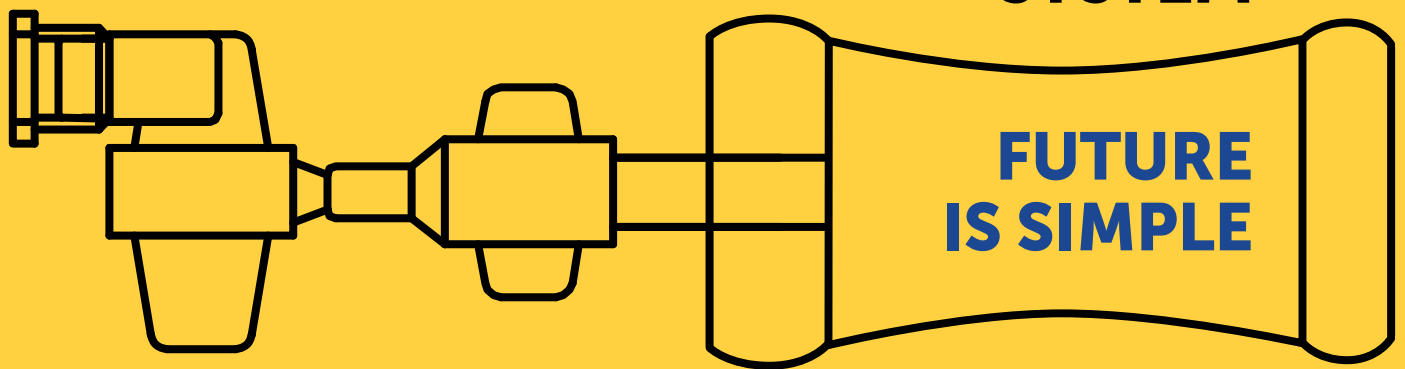

SERUM EYE DROPS
NEW PREPARATION SYSTEM

Distributed by LABEX
www.labex.com


BiomedDevice

PRODUCING SERUM EYE DROPS HAS NEVER BEEN SO EASY.

We developed a solution for preparation and preservation of serum eye drops in a quick and easy way.



By combining YDB with COL, we created Eye Drop System, a closed circuit system for production, preservation and application of serum eye drops.

Eye Drop System is suitable for all stages of the work cycle: from the vein to the patient's eye, in a sterile and quick way.

**SO
QUICK.**

**SO
SIMPLE.**





EYE DROP SYSTEM **YDB**

THE EASY TO USE AND CERTIFIED
DEVICE THAT REDUCES SERUM EYE
DROPS PRODUCTION TIME.

The presence of **special glass beads** accelerates coagulation, whereas the **anti-bacterial filter** ensures sterility of the finished product.

The device's configuration allows the **functionally closed circuit processing** so that there is no need for Clean Room or laminar flow cabinet.

YDB allows you to prepare serum eye drops with **reduced volume blood draw.**

YDB gives the possibility to prepare **long term treatments in one section only.**

YDB increases serum production rate, reducing costs and time **without need for expensive machines.**



EYE DROP SYSTEM

COL[®]

DEVICE DESIGNED, MANUFACTURED AND PATENTED FOR THE APPLICATION OF SERUM EYE DROPS AND PLATELET CONCENTRATE.

Depending on the type of therapeutic protocol, physicians will find **the COL configuration best suiting their needs.**

COL allows filling in a functionally closed system **without need of clean room or laminar flow cabinet.**

COL is patented and CE 1936 certified for preservation and application of hemocomponents in ophthalmic field.

COL allows you to give your patient an **applicator ready to use.**

The device is developed for administrating drops always identical and for **ensure sterility** thanks to the anti-bacterial filter.

To make easier and safer delivery of the applicators to the patient, Biomed Device supply a dedicated container.

YDB

REF	BAG FILLING VOLUME	SERUM VOLUME (WITHOUT DILUTION)
YDB150	0 - 150 ml	0 - 100 ml

COL®

REF	APPLICATORS PER KIT	ONE APPLICATOR FILLING VOLUME	DROPS PER APPLICATOR
COL10	10	0 - 1,45 ml	45
COL20	20	0 - 1,45 ml	45
COLB10	10	0 - 2,50 ml	80
COLB20	20	0 - 2,50 ml	80

COL®.C

REF	APPLICATORS PER KIT	ONE APPLICATOR FILLING VOLUME	DROPS PER APPLICATOR
COLC30	30	0 - 1,45 ml	45
COLC40	40	0 - 1,45 ml	45
COLC50	50	0 - 1,45 ml	45
COLCB30	30	0 - 2,50 ml	80
COLCB40	40	0 - 2,50 ml	80
COLCB50	50	0 - 2,50 ml	80

Distributed by LABEX
sales@labex.com
www.labex.com



BIOMED DEVICE S.R.L.
via Vittorio Bottego, 239 / 41126 Modena / Italy
tel. +39.059.343929 / fax +39.059.4550010
e.mail info@biomeddevice.it / www.biomeddevice.it

All products are **CE 1936** certified in accordance with European Directives.

Biomed Device is a certified company:



ISO 9001:2015



EN ISO 13485:2016

Management System
ISO 9001:2015
EN ISO 13485:2016
www.tuv.com
ID 9105080839



Biomed Device products are medical devices and not available to the general public, and must be used only by authorized personnel. Biomed Device reserves the right to change the products at any time, in line with technical innovation and product improvement. This informative material is for the exclusive use of authorized personnel.